

Busch[®]

New Product Bulletin

32HG 012 Amalgam Remover

Busch
**Quicker & Easier Removal
Of Amalgam Fillings, Is
Now Possible With The New**
32HG⁰¹²

This New Amalgam Remover from Busch[®] has transformed what was often an arduous and time-consuming task with diamond instruments or conventional carbide burs.

*FG-shank with blue ring
for easy identification*



The 32HG⁰¹² features:

- The finest grain carbide
- Special cutting geometry that reduces heat
- Design that allows both cross & longitudinal-section cuts
- Unique blade ensures effective cutting with no smearing

For additional information, contact

Pfingst
& Company, Inc.

*Busch is a registered trademark of Pfingst & Company, Inc.

105 Snyder Road, South Plainfield, NJ 07080
Phone: 908-561-6400 • Fax: 908-561-3213
E-mail: pfingst.co@rcn.com

As advertised in Dental Products Report, Dentistry Today & Oral Health

More Efficient At Removing Amalgam Than Either Diamond Instruments or Conventional Carbide Burs.

*For other burs within the line,
reference page 7 of our catalog.*

Made of the finest grain carbide, Busch's New 32HG 012 bur features a unique cutting geometry specially designed to remove amalgam with less generated heat and clogging, thus reducing smears. When recommended speed and pressure force is applied, the removal of existing fillings can be accomplished quickly and efficiently with the 32HG.



Recommended Speed:	160.000 to 180.000 rpm
Recommended Water Cooling:	at least 50ml/min.
Maximum Pressure Force:	2N
Bur Shank Diameter:	1.6mm
Sterilization:	All conventional methods of disinfecting & sterilization

*For additional information, or to place an order,
contact Pfingst & Company at 908-561-6400*

All orders received before 2:00 PM EST are shipped the same day.